



ENRICH[®]

OTC HEARING AIDS



**MANUAL
ENGLISH**

Welcome

We're thrilled to help you enjoy more of everything you love again.

Please read these instructions first to tap into the full potential of your device.

Thank you for choosing Enrich®

WARNING: If you are younger than 18, do not use this.

You should talk to a doctor, preferably an ear-nose-throat doctor (an ENT), because your condition needs specialized care. Over-the-counter hearing aids are only for users who are age 18 or older.

This OTC hearing aid is for users who are 18 and older. People who are younger than 18 with hearing loss should see a doctor, preferably an ENT, because they may need medical testing and management. Hearing loss can affect speech and learning, so professional fitting and continuing care are also important.

WARNING: When to See a Doctor

If you have any of the problems listed below, please see a doctor, preferably an ear-nose-throat doctor (an ENT).

- Your ear has a birth defect or an unusual shape.
- Your ear was injured or deformed in an accident.
- You saw blood, puss, or fluid coming out of your ear in the past 6 months.
- Your ear feels painful or uncomfortable.
- You have a lot of ear wax, or you think something could be in your ear.
- You get really dizzy or have a feeling of spinning or swaying (called vertigo).
- Your hearing changed suddenly in the past 6 months.
- Your hearing changes: it gets worse then gets better again.
- You have worse hearing in one ear.
- You hear ringing or buzzing in only one ear.

WARNING: This hearing aid should not cause pain when inserting it.

Remove this device from your ear if it causes pain or discomfort when you insert or place it. To try again, make sure to follow the instructions. If you feel pain or discomfort again, contact the manufacturer. If your pain or discomfort doesn't go away, contact your hearing healthcare professional. You can also report this to FDA as an adverse event according to the instructions that appear later.

Caution: This is not hearing protection.

You should remove this device if you experience overly loud sounds, whether short or long-lasting. If you're in a loud place, you should use the right kind of hearing protection instead of wearing this device. In general, if you would use ear plugs in a loud place, you should remove this device and use ear plugs.

Caution: The sound output should not be uncomfortable or painful.

You should turn down the volume or remove the device if the sound output is uncomfortably loud or painful. If you consistently need to turn the volume down, you may need to further adjust your device.

Caution: You might need medical help if a piece gets stuck in your ear.

If any part of your hearing aid, like the eartip, gets stuck in your ear, and you can't easily remove it with your fingers, get medical help as soon as you can. You should not try to use tweezers or cotton swabs because they can push the part farther into your ear, injuring your eardrum or ear canal, possibly seriously.

Note: If you remain concerned, consult a professional.

If you try this device and continue to struggle

with or remain concerned about your hearing, you should consult with a hearing healthcare professional.

Note: What you might expect when you start using a hearing aid.

A hearing aid can benefit many people with hearing loss. However, you should know it will not restore normal hearing, and you may still have some difficulty hearing over noise. Further, a hearing aid will not prevent or improve a medical condition that causes hearing loss.

People who start using hearing aids sometimes need a few weeks to get used to them. Similarly, many people find that training or counseling can help them get more out of their devices.

If you have hearing loss in both ears, you might get more out of using hearing aids in both, especially

in situations that make you tired from listening—for example, noisy environments.

Note: Tell FDA about injuries, malfunctions, or other adverse events.

To report a problem involving your hearing aid, you should submit information to FDA as soon as possible after the problem. FDA calls them “adverse events,” and they might include: skin irritation in your ear, injury from the device (like cuts or scratches, or burns from an overheated battery), pieces of the device getting stuck in your ear, suddenly worsening hearing loss from using the device, etc.

Instructions for reporting are available at <https://www.fda.gov/Safety/MedWatch>, or call 1-800-FDA-1088. You can also download a form to mail to FDA.

CONTENTS

What’s Inside	8
Getting to Know Enrich®.....	9
Wearing Your Hearing Aids.....	10–11
Operation	12
Battery Information.....	12
How to Adjust Volume	13
Profiles	14
Profile Selection.....	15
General Product Care.....	16–19
Troubleshooting.....	20–21
Technical Data.....	22
Warranty	23
Battery Safety	24–25

What's Inside

- 1 pair of Enrich® Hearing Aids
- 3 pair Assorted Tips
- 2 pair Sound Tubes
- Hard Case
- Cleaning Tool
- #13 Batteries
- User Manual



Hard Case



Enrich®
Hearing Aids



x3

Assorted Tips



x4

Batteries



Cleaning Tool



x4

Sound Tube

Getting To Know Enrich®



Wearing Your Hearing Aids

1



Place the hearing aids on the ear. Insert the appropriate size dome tip onto the sound tube at the elbow joint then insert the dome tip into ear.


2



Position the sound tube in an upward direction so that it extends past the ear hook.


Mark the sound tube approximately 1/8" above the tip of the ear hook.

3

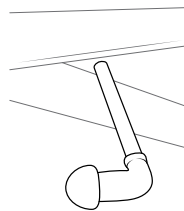

Cut off excess tube at mark. Tubing too long will cause the unit to fall out. Tubing too short will pull the tip out of the ear.

4

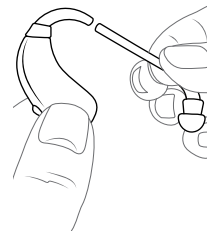

While holding the sound tube between thumb and forefinger, push the sound tube onto the ear hook. You may need to trim the tube and repeat this process to achieve a comfortable fit.

Tip: Remove unit from ear gently to avoid damaging tube.

Operation

POWER ON: Close battery door with battery inserted

POWER OFF: Open battery door

Battery Information

Low Battery voice prompt indicates it is time to replace the battery. The batteries must be inserted correctly or the hearing aids will not work.

INSERTION: Remove adhesive sticker on #13 battery. Open battery door and place battery with + side upward. Closing the door will activate the unit in Program 1.

REMOVAL: Open the battery door and lift the #13 battery out.

Expected battery life: 12 days



How To Adjust Volume

1. Press volume selector **UP** to raise volume
2. Press volume selector **DOWN** to lower volume

You will hear a quick double tone indicating volume has changed when pressing.



Tip: Press and release, do not hold down

Profiles

There are 3 profiles to choose from:

- 1 General Listening:**
Amplify higher frequency sounds and in most situations provide the most comfortable listening.
- 2 Crowds:**
Amplify high frequency sounds in noisy situations.
One tone prompt.
- 3 Music/Concert -**
Amplify low frequency sounds.
One tone prompt.

Profile Selection

Hold toggle switch **UP/DOWN** for 2 seconds to cycle through profiles. A two-beep prompt will announce the profile change. Press and hold again to cycle through the profiles.

You should evaluate the different profiles for each listening environment to determine which is the best, most comfortable sound.

Toggle Switch



General Product Care

The most common cause of instrument failure is wax buildup and it is the easiest to avoid. By wiping your hearing aids with a soft, dry cloth every day and carefully removing any traces of wax and debris, you will reduce wax buildup. A brush is provided to further assist you in cleaning your device.

Use the brush provided to enhance your cleaning by lightly brushing the sound ports. It is important to keep your hearing aids as clean as possible to maintain maximum performance and durability.



Cleaning
Tool

To clean your ear tips, remove them from the hearing aid and then wash in lukewarm water with a mild detergent. Dry thoroughly before reconnecting.

Remember to remove the hearing aids before showering or bathing to avoid water damage to the devices.

Your hearing aids are delicate, precision electronic devices and require special care and handling that should be provided to such devices. Please read and follow the listed precautions to ensure quality performance and long instrument life:

- Remove batteries when hearing aids will not be in use for prolonged periods of time.
- Do not throw or drop your hearing aids.
- Do not immerse hearing aids in liquid.
- Do not wear during bathing, showering or swimming
- Store out of the reach of children and pets.
- Do not expose hearing aids to excessive heat, such as stove top, microwave, dash board of a hot car, or hair dryer.
- Do not expose the hearing aids to chemicals such as hair spray, insect repellent, perfume, or cosmetics.

General Product Care (cont'd)

- Do not use the product with higher volume for a long period of time, in order to avoid hearing damage. Stop using if discomfort occurs.
- Do not disassemble the device
- To avoid possible electromagnetic radiation, the devices should be removed in any high frequency, heat treatment or operation, radiotherapy and strong magnetic field environments.
- Please avoid use in a potentially explosive or dangerous situation.
- Only qualified accessories should be used with this device.
- Do not wear the hearing aids while sleeping.

- Please keep the surface of hearing aids clean, clean wax build up in tips.
- Please use the toggle switch correctly, avoid excessive force as far as possible.

Special Note: Your hearing aids are built with the finest quality materials. However, in rare instances an individual may experience sensitivity to the materials used. Please consult with your physician if you exhibit any signs of skin irritation, allergic reaction, or accelerated accumulation of cerumen (wax).



Important: Keep your hearing aids as clean as possible to maintain maximum performance.

Troubleshooting

ISSUE	POSSIBLE CAUSE	SOLUTIONS
No/Low sound output	Low battery	Replace Battery
	Volume set too low	Increase the volume to a comfortable level
	Tip not inserted properly	Re-insert the tip
	Hearing aid is damaged	Contact Customer Service for assistance
	A significant change in your hearing	Contact your hearing health care professional for an exam. Engage Customer Service can recommend a specialist in your area

Still have issues? Reach out to us at **866-202-2527** or chat with us at **lucidhearing.com** and our specialists will be happy to help you.


Your satisfaction is our number one priority!

ISSUE	POSSIBLE CAUSE	SOLUTIONS
Whistling/ Feedback	Tip not inserted properly	Reinsert hearing aid tip
	Dome tip damaged or is the wrong size	Replace dome tip with a new tip
	Wax build-up in the ear canal	Talk to your doctor or a hearing healthcare professional about how to best clean your ears
Hearing aids not turning off	Battery door closed	Open battery door
Device is not comfortable	Tips too large or too small	Try a different size dome tip
	Tube is too large or too small	See Tube Fit Guide (included)

Technical Data

Warranty

ANSI TEST

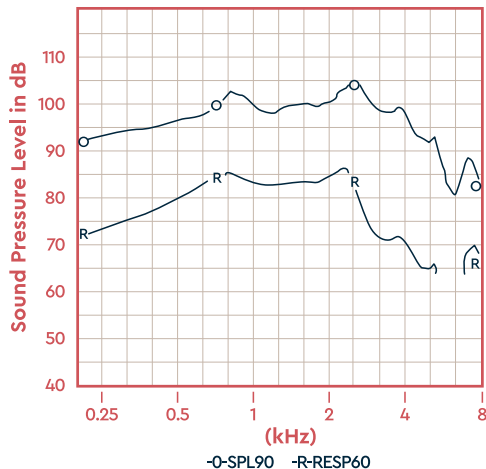

Linear FOG@50 dB
OSPL90 Max 105.4 dB
@ 2500 Hz
HFA 101.0 dB
HFA FOG @ 50 23.7 dB
RTG TARGET 24.9 dB
Measured 23.8 dB
EQ Insp noise 28.2 dB
Resp limit 63.7 dB
F1 < 200 Hz F2 > 8000 Hz

HFA 1000, 1600, 2500 Hz		
Freq	THDist	Src
500Hz	0.1%	70 dB
800Hz	0.2%	70 dB
1600Hz	0.2%	65 dB

Lucid Hearing OTC products are risk free with a 60 day return period and a 1 year manufacturer's warranty. To file a claim please call us at 866-458-8947.



Warranty Warning:

Please do not attempt to make repairs yourself. Doing so could cause additional damage and will void any warranties.

- **Do not throw** or drop your instruments.
- **Do not immerse** hearing instruments in liquid or water.
- **Do not wear** during bathing, showering, or swimming.
- **Do not expose** instruments to excessive heat, such as a stove top, microwave, dashboard of a hot car, or hair dryer.
- **Do not expose** the instruments to chemicals such as hair spray, insect repellent, perfume, or cosmetics.
- **Do not disassemble** the device.
- **Avoid use in potential explosive** or dangerous situations.

Battery Safety



CAUTION – Battery Safety Information Hearing Aid Batteries are Dangerous if Swallowed

Keep hearing aids out of the reach of children or anyone who may accidentally swallow them. Never put hearing aids in your mouth for any reason. Do not store in pill containers where they may be mistaken for medication and swallowed. Do not attempt to remove batteries from your device, see your hearing care professional. Do not use any other rechargeable battery; Batteries of low quality may leak and cause bodily harm. Batteries with incorrect voltage may cause damage to your device. They may explode and cause serious injury. Do not expose your hearing aid to heat, such as an open flame, microwave, or oven. There is a risk that they will explode and cause serious injury.

Visit <http://www.poison.org/prevent/battery.asp> to learn more important information about battery safety.

If a battery or hearing instrument is swallowed, see a doctor immediately and call the National Capital Poison Center, 202-625-3333.

Environmental Conditions

Storage and Transport: 10° to 30°C

Charge Conditions: 0° to 45°C

Operational Conditions: 0° to 50°C



Disposal of Instruments
Waste from electronic equipment must be handled according to local regulations.



WARNING: Risk of Fire and Burns. Do not Open, Crush, Heat above 60°C/131°F or incinerate. Do not short circuit. If bulges severely, discontinue use.



Manufactured by:

Xiamen Wenatone Medical
Technology Co. Ltd
A805 Jianye Building, Torch Hi-Tech
(Xiang An) Industrial Park
Xiamen, Fujian, China

Distributed by:

Lucid Hearing Holding
Company, LLC
PO BOX 535596
Grand Prairie, TX 75053

Transport & Storage



Operational Conditions



Type B applied part



Read Instructions



lucidhearing.com

©2022 Lucid Hearing Holding Company, LLC. Lucid Hearing and Enrich are registered trademarks of Lucid Hearing Holding Company, LLC. All rights reserved.